

WOMEN

(Men use other side)

Quick Reference Guide

IFU available at: I-Ourology.com




Overview

The CarePath® system measures urine output and allows tracking of associated data through the mobile application.



Mobile App Setup

Tap on the link that you received in your text messages from CarePath. For your phone, follow these steps:

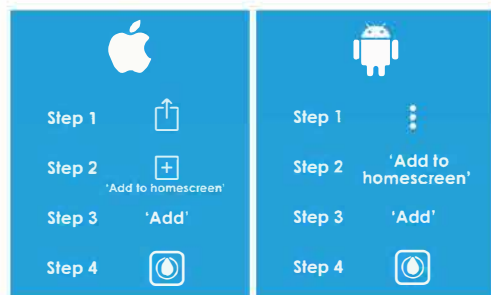

Consult your healthcare provider if you have issues. You may need to enter your phone number for setup.

ClearTrac Technologies, LLC
730 Quail Hollow Dr., Elizabethton, TN 37643

For questions or technical support, call:
423-647-2222, M-F, 8-5 EST

Urinating into the Device


Your healthcare provider relies on the information provided by this system.

Use this device EVERY time you urinate, as instructed.

1

Sit on the toilet and hold the device as shown.

Keep the device against your body.

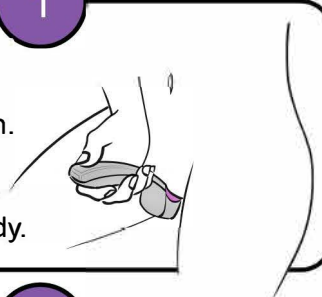

2

Device must be correctly oriented during urination.

Green light: Correct orientation

Yellow light: Adjust angle of device until light turns green

Red light: Contact your care provider

3

When finished urinating, lift handle to tilt device, allowing it to drain completely.

Clean the device by rinsing it with water and storing in the pouch between uses.

System Use

The CarePath® system provides greater accuracy for tracking urine output and associated data.

There are no known risks when the device is used as instructed.

Contact your healthcare provider if the system is not working as expected or was used in a way other than instructed.

Cautions and Warnings

Caution: To clean your CarePath® device, rinse it in water only. Do not use any other method of cleaning since it may affect the accuracy of the device.

Warning: To avoid the risk of fire, electric shock, or burns, do not crush, incinerate, put in dishwasher, put in microwave, or heat the device above 212°F (100°C). Do not store the CarePath® device in your car or in any environment where excessive heat is possible.

Caution: Do not use the device in temperatures outside the range of 50° to 104°F (10° to 40°C), since it may not provide accurate data.

MEN

(Women use other side)

Quick Reference Guide

IFU available at: I-Ourology.com



Overview

The CarePath® system measures urine output and allows tracking of associated data through the mobile application.



Mobile App Setup

Tap on the link that you received in your text messages from CarePath. For your phone, follow these steps:

Step 1	Step 1
Step 2 'Add to homescreen'	Step 2 'Add to homescreen'
Step 3 'Add'	Step 3 'Add'
Step 4	Step 4

Consult your healthcare provider if you have issues. You may need to enter your phone number for setup.

ClearTrac Technologies, LLC
730 Quail Hollow Dr., Elizabethton, TN 37643

For questions or technical support, call:
423-647-2222, M-F, 8-5 EST

Urinating into the Device


Your healthcare provider relies on the information provided by this system.

Use this device EVERY time you urinate, as instructed.

1

Sit on the toilet and hold the device as shown.

Sitting gives the best flow data.

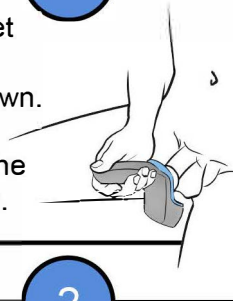

2

Device must be correctly oriented during urination.

Green light: Correct orientation

Yellow light: Adjust angle of device until light turns green

Red light: Contact your care provider

3

When finished urinating, lift handle to tilt device, allowing it to drain completely.

Clean the device by rinsing it with water, and storing in the pouch between uses.

System Use

The CarePath® system provides greater accuracy for tracking urine output and associated data.

There are no known risks when the device is used as instructed.

Contact your healthcare provider if the system is not working as expected or was used in a way other than instructed.

Cautions and Warnings

Caution: To clean your CarePath® device, rinse it in water only. Do not use any other method of cleaning since it may affect the accuracy of the device.

Warning: To avoid the risk of fire, electric shock, or burns, do not crush, incinerate, put in dishwasher, put in microwave, or heat the device above 212°F (100°C). Do not store the CarePath® device in your car or in any environment where excessive heat is possible.

Caution: Do not use the device in temperatures outside the range of 50° to 104°F (10° to 40°C), since it may not provide accurate data.